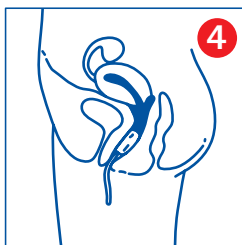
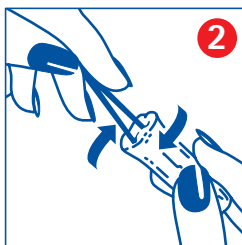
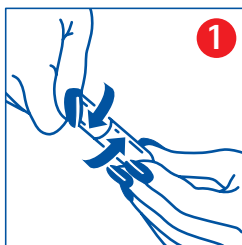
# senta

## TAMPONS

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**Please read the directions  
and health advice!**

### **Senta Tampons offer you three-way comfort:**

1. The soft, rounded tip allows the tampon to be inserted easily and gently.
2. The special grooved structure means that the fluid is quickly drawn inside the tampon – the tampon is able to expand evenly, gently fitting the contours of your body.
3. The soft fleecy cover ensures that the tampon keeps its shape even while it is being removed.

### **Senta Tampons come in four sizes:**

- **Mini Comfort:** for “lighter” days and for young girls
- **Normal:** for normal to moderately heavy menstruation
- **Super:** for heavy menstruation
- **Super Plus:** for a very heavy menstrual flow and after birth

Please, always use the smallest size suitable.

### **How to use Senta tampons correctly:**

First, please wash your hands thoroughly. Twist the lower and upper parts of the cellophane cover in opposite directions as indicated by the arrows [Figure ①]. Pull off the lower part of the cover where you see the removal tape. After drawing out the removal cord, widen the tip of the tampon using circular movements to give a small recess [Figure ②]. Now place the tip of your index finger in the recess in the tip of the tampon and remove the top of the cellophane wrapper [Figure ③]. Insert the tampon into the vagina until you are no longer able to feel it. Make sure the removal cord is hanging far enough outside the vagina for you to be able to grip it firmly [Figure ④].

### **How to remove the Senta tampon:**

Just ease the tampon out at an angle, using the removal cord. This is easiest to do when the tampon is saturated.

### **Note on wearing time:**

When the blood flow is at its heaviest, it is best to change the tampon every **3 to 4 hours**. As the flow gets lighter, or at **night**, the tampon can be left in the vagina for longer. However, do not exceed **6 to 8 hours**. Never use more than one tampon at a time. We advise you to check that the tampon is removed when your menstruation is over.

### **Important health advice:**

In rare cases, symptoms may occur during menstruation such as a high temperature above 39° C, vomiting, diarrhoea, dizziness or rashes. Although some harmless illnesses start in just the same way, a particular type of bacterial infection (TSS: Toxic Shock Syndrome) may be life-threatening.

If any of the symptoms described occur, please remove the tampon, consult a doctor immediately and tell him/her that you are menstruating. We advise women who have previously suffered from TSS to consult their doctor before using tampons again.

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