senta protect

BADE-
UND
SCHUTZ
TAMPONS
PROTECTION
TAMPONS

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GB **IMPORTANT: Please read the instructions for use and health advice.**

If you are using Senta PROTECT bathing and protective tampons for the first time, you will find important information in the instructions for use.

Senta PROTECT bathing and protective tampons have been specially developed for women who place value on hygiene when bathing. A special bathing and protective tampon has therefore been developed that protects the woman's genital area when bathing and swimming, and reduces the entry of e.g. chlorinated or potentially polluted water. With this protection the area which bacteria and fungi may attack in the vagina is also reduced.

Senta PROTECT bathing and protective tampons offer you security and comfort.

- The tampon is impregnated with pure vaseline oil which has a waterrepellent effect and thus prevents water from entering the vagina.
- The size of the Senta PROTECT bathing and protective tampons is the same as the

“normal” size of conventional tampons.

- The soft rounded tip allows the tampon to be inserted easily and gently.
- The soft fleecy cover ensures that the tampon retains its shape even when being removed.
- You can also use Senta PROTECT bathing and protective tampons as the period is coming to an end. They are easy to insert and just as easy to remove.
- Senta PROTECT can also be used by healthy pregnant women whose pregnancy is proceeding normally.

How to use Senta PROTECT bathing and protective tampons correctly:

First, please wash your hands thoroughly. Open the pack along the perforation, remove the foil and press the tampon out of the pack. Remove the tampon together with its removal cord. After unwinding the removal cord, widen the tip of the tampon using circular movements to make a small recess. Now place the tip of your index finger into the recess in the tip of the tampon. Guide the tampon into the vagina until you are no longer able to feel it. Take care to ensure that the removal cord is hanging far enough outside the vagina to allow you to grip it firmly.

Note on wearing time:

We recommend inserting the tampon before bathing or swimming, and it is best to change it every three to four hours.

How to remove the Senta PROTECT bathing and protective tampon:

Simply pull the tampon carefully forwards by the removal cord, and then downwards to remove it.

Note on storage:

Please protect the product from exposure to direct sunlight. Keep in a cool, dry place.

Important health advice:

In rare cases, symptoms such as high fever of over 39° C, vomiting, diarrhoea, dizziness or a skin rash may occur during menstruation. Although a number of harmless illnesses also start in this way, a very specific bacterial infection (TSS: toxic shock syndrome) can have serious implications. If the symptoms described appear, please remove the tampon and seek medical advice, telling the doctor that you are menstruating.

Personal advice:

Please write to us if you have any questions. Our female staff at Senta Customer Advice will reply personally and discreetly to each letter.

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